

NARCO SPRAY®

COCAINE CRACK

- STEP I TEAR ONE COLLECTION CARD FROM THE PAPER DISPENSER.
STEP II WIPE THE SUSPECTED AREA WITH THE STICKY SIDE OF THE PAPER OR PLACE A SMALL AMOUNT OF THE SUBSTANCE ON THE COLLECTION CARD.
- STEP III SHAKE THE AEROSOL CANISTER WELL.
STEP IV SPRAY THE PAPER BRIEFLY FROM A DISTANCE OF 10 CM (4").
STEP V WHEN PRESUMPTIVELY POSITIVE, A BLUE / TURQUOISE COLOR WILL APPEAR.

COCAINE	STRONG BLUE IMMEDIATELY
PCP	STRONG BLUE WITHIN 10 SECONDS
METHAQUALONE	STRONG BLUE WITHIN 10 SECONDS
GBL	MEDIUM BLUE WITHIN 10 SECONDS
MDPV	BLUE WITHIN 10 SECONDS
EPHEDRINE	BLUE WITHIN 10 SECONDS
CRACK	BLUE SPOTS WITHIN 10 SECONDS
BUPRENORPHINE	BLUE SPOTS WITHIN 10 SECONDS
METHADONE	BLUE SPOTS WITHIN 10 SECONDS



STRONG BLUE
(COCAINE HCL)



BLUE
(GBL)



BLUE SPOTS
(CRACK)

IF A POSITIVE REACTION OCCURS, SUBMIT THE EVIDENCE FOR FURTHER LABORATORY ANALYSIS ACCORDING TO YOUR AGENCY'S PROTOCOL.



M.M.C. INTERNATIONAL B.V.
FRANKENTHALERSTRAAT 16-18
4816 KA - BREDA - THE NETHERLANDS
+31-76-5711140 - INFO@MMCINTER.COM

PRODUCT NO. MMC-NSCOC

NARCO SPRAY®

GENERAL SCREENING


- STEP I TEAR ONE COLLECTION CARD FROM THE PAPER DISPENSER.
- STEP II WIPE THE SUSPECTED AREA WITH THE STICKY SIDE OF THE PAPER OR PLACE A SMALL AMOUNT OF THE SUBSTANCE ON THE COLLECTION CARD.
- STEP III SHAKE THE AEROSOL CANISTER WELL.
- STEP IV SPRAY THE PAPER BRIEFLY FROM A DISTANCE OF 10 CM (4").
- STEP V OBSERVE FOR A COLOR OTHER THAN THE LIGHT YELLOW BACKGROUND COLOR OF THE REAGENT.
- STEP VI PERFORM A BLANK TEST IF YOU'RE NOT FAMILIAR WITH THE BACKGROUND COLOR. COMPARE THE COLOR TO THE CHART ON THE LABEL OF THIS CANISTER. IF THE COLOR MATCHES THEN THE TEST IS POSITIVE FOR THIS SUBSTANCE.




BLANK (ORIGINAL)



KETAMINE

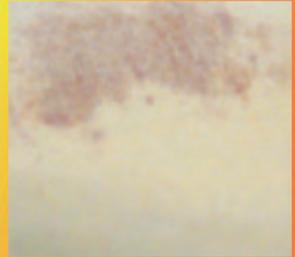

HEROIN / OPIUM

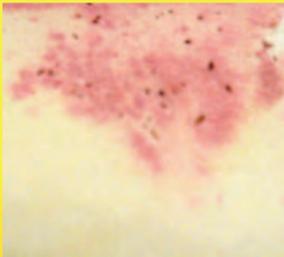

CANNABIS



METHAMPHETAMINE / MDMA



PCP

IF A POSITIVE REACTION OCCURS, SUBMIT THE EVIDENCE FOR FURTHER LABORATORY ANALYSIS ACCORDING TO YOUR AGENCY'S PROTOCOL.



M.M.C. INTERNATIONAL B.V.
FRANKENTHALERSTRAAT 16-18
4816 KA BREDA THE NETHERLANDS
+31-76-5711140 - INFO@MMCINTER.COM

PRODUCT NO. MMC-NSGS