



MMC INTERNATIONAL

Your partner in the fight against drugs...

2013-2014

IN THIS ISSUE

INTRODUCTION

MMC International BV is the manufacturer of the MMC presumptive narcotic field test kits and supplier of many related products.

We have our own laboratory where we conduct analysis (GC/MS etc.) and develop new MMC test kits.

The major clients are: Police, Customs & Excise, Prisons and other Law Enforcement Authorities.

We have a distributor network worldwide of approx...50 distributors

In this newsletter we want to inform you about our product profile.

FREE PORTABLE DRUG SCALE!

When ordering 50 MMC narcotic test packs or more (mix allowed) we will add one Portable Drug Scale for free. (Max. 1 per new customer)



COMPANY PROFILE

M.M.C. International. B.V. established in 1992, located in Breda, The Netherlands.

M.M.C. is specialized in the production of Narcotics- and Precursor Identification - Tests. The Narcotics- and Precursors - Tests are developed in close cooperation with Police and Customs & Excise authorities. The company is certified by VERITAS and awarded with the EN ISO 9001-2000 certificate.

The company is registered as a vendor since 1994 with all UN Organizations under No.100346 / Net 0309 and UN Global Market place # 149548.

M.M.C. International's laboratory is especially focused on the production and development of narcotic identification products. The tests are produced in accordance with the requirements set by Police Forces, Customs & Excise and Forensic Laboratories.

The ampoules are filled with special treated solid materials which are impregnated with very high quality reagents and a blanket of high purity nitrogen and hermetically sealed. Each batch is continuously tested and registered.

All products are stamped with a batch number, production and expiry date.

MMC also supplies many other related products such as the MMC Narcotics Education Case, DOA tests, Mobile Narcotics Lab Trolley, Inspection Cameras for customs etc...

Introduction




MMC Narcotic Field Test Kits and how to use them.



OLD NEW

MMC Synthetic Cannabinoids Test

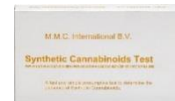

MMC Specialty Tests



A closer look at the MMC Forensic Tests



Other Products and Training Courses





MMC Field Tests

Over the years MMC International developed many presumptive field tests. These tests are mainly used by **law enforcement agencies, government agencies and crime labs.**

Unlike other presumptive drug testing kits, the MMC International kits contain very little reagents and do not require the use of an acid neutralizer. This revolutionary new design eliminates any potential danger to officers when using these kits in the field.

FEATURES

- Safe, easy to use and economical
- No environmental risks/concerns
- One step tests (one ampoule only)
- Results within seconds
- High reliability and consistency
- Accurate and sensitive
- Easy to test for drugs in liquid form such as GHB, GBL, and liquid XTC, etc.



**Not to test drinks*

Also available: 1,4BD/GHB/GBL Test Strips



We also have many Narcotic test cases available with or without tools. We also can supply empty carrying cases in different sizes...

MMC Narcotic Field Tests

- Barbiturates Test
- Buprenorphine Test
- Cannabis / Marijuana Test
- Cocaine / Crack Test
- Crystal Meth / XTC Test
- Ephedrine (HCL) Test
- Fentanyl/Oxycodone/ Heroin-Morphine and Codeine Test
- Dried Qat (khat) Test
- Fresh Qat (khat) Test
- General Screening Test
- GHB Test
- Heroin Test
- Ketamine Test
- LSD Test
- M&H Test
- mCPP Test
- Methadone Test
- Methaqualone Test
- Opiates/Amphetamines Test
- PCP Test
- Pentazocine Test
- Phenethylamines Test (2C's - Smiles)
- Methamphetamine / MDMA Test
- Prime Test
- Propoxyphene (PPX) Test
- Synthetic Cannabinoids Test (K2, Spice)
- Synthetic Cathinones Test (Bath Salts)
- V&R (Valium/Rohypnol) Test
- Cocaine Odor Test Kit
- Various Test

Many tests have been evaluated/approved by the NFSTC and Virginia Science Lab.

MMC Precursor Field Tests

- Calcium hydroxide test
- Acetone test
- Acetic anhydride test
- Anthracitic acid test
- Ammonium chloride test
- Ergotamine test
- Ergometrine / Oxycotin test
- (Iso)-safrole test
- 1-phenyl-2-propanone test
- Phenyl acetic acid test
- Piperonal test
- Potassium permanganate test
- Propoxyphene HCL test
- Sodium carbonate test
- Toluene test
- GBL test
- Ephedrine test
- PMK/BMK Test
- Apan Test

MMC Specialty Tests

- Cocaine Purity Test
- Heroin Purity Test
- MDMA purity Test
- Cocaine Analyzing Kit
- Cocaine Cutting Agents Test



New designed test ampoule with ampoule breaker on top of each test for easy and safe usage.

How to use

1



Knock the unopened ampoule several times against a hard surface, so the granules which may be sticking to the inside of the ampoule wall will fall downwards.

2



Take the top of the ampoule between the thumb and index finger and snap off the top with the ampoule breaker.

3



Use the spatula to insert the suspect material into the ampoule.

4



For suspect samples in liquid form such as GHB, we advise to use the round end of the supplied spatula or otherwise use a pipette and insert one (1) drop into the ampoule.

5



With some of the MMC tests, the plastic lid contains a catalyst pearl which, once the material to be tested is inserted in the ampoule, must be mixed using the round tip of the spatula.

6



Attention! If the ball in the cap sticks against the side, please use a spatula to take it out and, put it in the ampoule and mix it. Place the plastic cap on top of the ampoule.

7




Compare the color from the reagent granulate with the color line printed on the ampoule or as printed on the pack.

8



Since the MMC Tests does not require acid neutralizers (as this is automatically done within the ampoule) you may throw the used ampoules in the trash or glass bin.

NEW DESIGN FOR YOUR SAFETY!

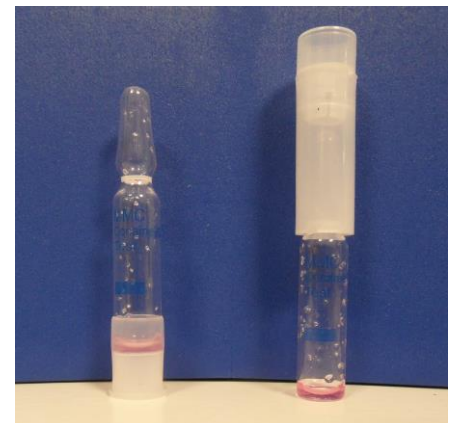

OLD NEW

MMC SYNTHETIC CANNABINOIDS TEST

There are hundreds of analogs and therefore it would be impossible to identify many of them with just one (1) colorimetric test. The test we developed is able to identify the below substances with their color reactions:

HOW TO USE THE TEST PROPERLY

- 1: Break off the top with the plastic ampoule-breaker on top of the ampoule
- 2: Insert a small quantity of the suspected Material 1/5 of the supplied spatula
- 3: Shake the ampoule twice! (this means 2 times a quick shake from left to right)
- 4: Hold the ampoule vertically. The top layer of the liquid inside the ampoule will be dark brown to black and the bottom layer will be as the colours mentioned. Compare the color of the liquid with the colors on the right →→→→

It could also happen that several active substances are mixed in one blister. In that case the color will most of the time be the same as the first color on the chart.



JWH-018, JWH-073, JHW-081 JWH-019, JWH-122, AM2201, XLR-11, AB-001, UR-144 Or a mixture of different cannabinoids	YELLOW / BROWN	
JWH-200, AM2233, AM2201, RCS-4 Or a mixture of different cannabinoids	YELLOW	
AM-694	GREY/BROWN	
CP-47,497	DARKBLUE, FADING OVER TIME	
CP-55,940	BLUE/GREEN, FADING OVER TIME	
URB-597	YELLOW CHANGING INTO GREY	
JWH-250	GREY	



MMC Lab Analysis

In this report we tested our Synthetic Cannabinoids test and we confirmed our samples with GC-MS.


Equipment Used

In our analysis of the pure substances used with our Synthetic Cannabinoids test we used our own GC-MS to confirm the identity of the samples. Below are the specifications of our equipment.



The GC was an Agilent Technologies 7820A Gas Chromatograph (GC) with 5975 Mass Selective Detector (MSD). The system uses a split/split less inlet with an HP5-MS column. Besides the electronic equipment we used regular laboratory supplies and our Synthetic Cannabinoids test.

GC-MS results

After we acquired our pure synthetic cannabinoids we confirmed through GC-MS that they indeed were the substance which the label said it was. For this purpose we ran a GC-MS analysis and subsequently ran the spectra through the designer drugs database 2013 and the National institute of standards and technology (NIST) database for confirmation. An example of the results for the synthetic cannabinoid JWH-018 is given below.

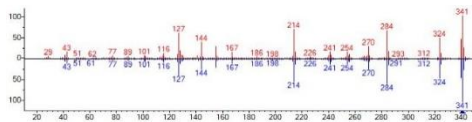

JWH-018 spectrum and the NIST database JWH-018 example spectrum

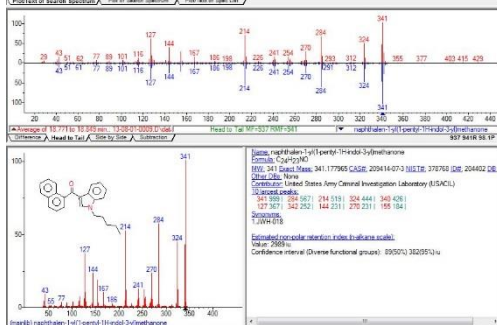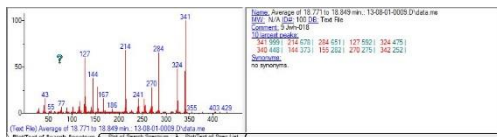


NIST database report on our JWH-018 spectrum.

The reports of the other synthetic cannabinoids were all similar, showing that our samples all contained the advertised synthetic cannabinoids with no to little impurities.

Results of the Synthetic Cannabinoids Test

We at M.M.C. International performed a test to determine exactly which color develops due to which synthetic cannabinoid in our Synthetic Cannabinoids test. Below you will find a picture of the test. This test was performed using the exact same substances whose identities were confirmed using GC-MS.

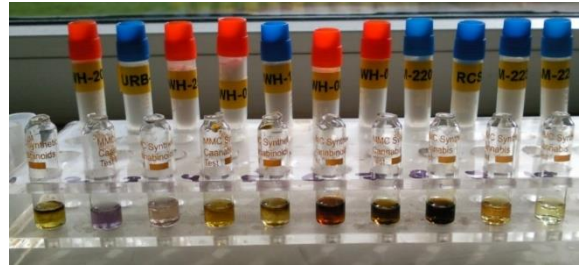

Analysis of different synthetic cannabinoids in our Laboratory..



Mixture of several synthetic cannabinoids in street sample (shaken twice!)

SPICE

A combination of cannabis-like synthetic chemicals sprayed on herbs and referred to generically as Spice



GC/MS Laboratory Analysis



Preparing a smoke of spice



Gas Station Counter sale of Synthetic Cannabinoids



Synthetic Cannabinoids in retail packaging.

MMC COCAINE PURITY TEST



This test we especially developed for undercover agents so they will be able to test cocaine on its purity. **The more pure, the closer to the source.....** Dealers often cut their merchandize in order to increase their profit margins. You will never know with what kind of substance and how much your coke has been cut by just looking at it.

The Cocaine Purity Test enables you to quickly get an idea of the purity of the cocaine. Just add 20 milligram of your sample to this test and you will have a result within seconds.

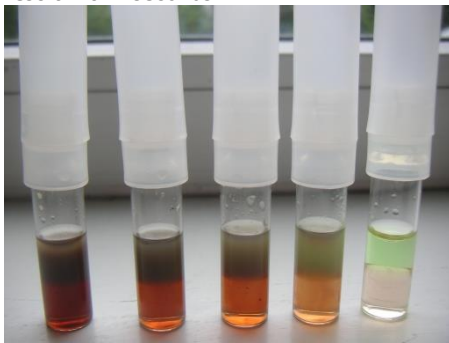


Image of the difference in purity of cocaine samples.

The test results above can be read as follows:

The first ampoule, where the bottom layer is almost like coffee was a **90%** pure sample and gets 5 stars!

The second ampoule was a **70%** pure sample and gets 4 stars.

The third ampoule was a **60%** pure sample and gets 3 to 4 stars.

The fourth ampoule is between 2 and 3 star quality (a **50%** pure sample, very common to find on the street!)

The last ampoule has almost no cocaine in it. It is not even worth 1 star (it was a **10%** pure sample).



The pack contains 10 Cocaine Purity Tests, 10 spatulas, 10 Buffer Solution Tubes and one instruction sheet.

Officers are now able to determine if the suspected substance is cocaine but also the purity of it.

The MMC Purity test is very insensitive to the usual cuts such as mannitol, but will as a bonus, not react to any other substance from the family of cocaine (lidocaine, procaine, etc.), which are frequently used to fool the soul who wants to taste it.

PROFESSIONAL POCKET WEIGHING SCALE

We recommend this professional pocket weighing scale together with the Cocaine Purity Test.



0.001 gr – 20 gr 95 x 33 x 133 mm



MMC Heroin Purity Test



After the successful launch of the MMC Cocaine Purity Test, we now managed to develop a Purity Test for Heroin.

We received many positive responses from the US market about the Cocaine Purity Test as it makes the task easier for the Officer to investigate how close they are to the source.

Officers were asking us if we also would be able to develop a test for the purity of Heroin.



The first ampoule, has almost no Heroin in it. It is not even worth one (1) star (it was a 10% pure sample)


The Second ampoule is worth 2-3 stars (it was a 40% pure sample)


The Third ampoule is worth 3-4 stars (it was a 60% pure sample)

The Fourth ampoule is worth 4 stars (it was a 70% pure sample)


The Fifth ampoule is worth 5 stars (it was a 85-90% pure sample)




 ★ (1 star) - Very low heroin content

 ★★ (2 stars) - Low heroin content

 ★★★ (3 stars) - Medium heroin content

 ★★★★ (4 stars) - High heroin content

 ★★★★★ (5 stars) - Very high heroin content

PROFESSIONAL POCKET WEIGHING SCALE

We recommend this professional pocket weighing scale together with the Heroin Purity Test.



0.001 gr – 20 gr

95 x 33 x 133 mm

MMC MDMA Purity Test



After the successful launch of the MMC Cocaine Purity Test, we now managed to develop a Purity Test for MDMA.

We received many positive responses from the US market about the Cocaine Purity Test as it makes the task easier for the Officer to investigate how close they are to the source.

Officers were asking us if we also would be able to develop a test for the purity of MDMA.



The first ampoule, has almost no MDMA in it. It is not even worth one (1) star (it was a 10% pure sample)






The Second ampoule is worth 2-3 stars (it was a 40% pure sample)

The Third ampoule is worth 3-4 stars (it was a 60% pure sample)

The Fourth ampoule is worth 4 stars (it was a 70% pure sample)

The Fifth ampoule is worth 5 stars (it was a 85-90% pure sample)



-  ★ (1 star) - Very low heroin content
-  ★★ (2 stars) - Low heroin content
-  ★★★ (3 stars) - Medium heroin content
-  ★★★★ (4 stars) - High heroin content
-  ★★★★★ (5 stars) - Very high heroin content

PROFESSIONAL POCKET WEIGHING SCALE

We recommend this professional pocket weighing scale together with the MDMA Purity Test.




0.001 gr – 20 gr

95 x 33 x 133 mm



MMC COCAINE CUTTING AGENTS TEST

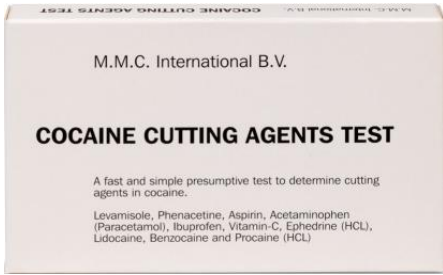

The MMC Cocaine Cutting Agents test is a quick presumptive test to see if the cocaine sample has been cut with adulterants.

Dealers often cut their cocaine to increase their profit margins.

The substances used for cutting are generally harmless but it sometimes happens that dangerous substances are added.

Instructions:

Break off the top with the plastic ampoule-breaker

Insert a small quantity of the cocaine into the ampoule

Stir the suspected substance with the supplied spatula

Compare the color of the liquid with the colors on the chart.

When presumptively positive the color should be:

Contents kit:

- * 10 test ampoules
- * 10 Spatulas
- * 1 Instruction Sheet



LEVAMISOLE	GREEN		EPHEDRINE (HCL)	YELLOW / GREEN	
ACETAMINOPHEN (Paracetamol)	OLIVE GREEN		LIDOCAINE	ORANGE / BROWN	
PHENACETINE					
ASPIRIN					
IBUPROFEN	DARK BROWN		BENZOCAINE	LIGHT BROWN	
VITAMIN-C	AQUA-BLUE		PROCAINE	ORANGE	



MMC COCAINE ANALYZING KIT



A **complete kit** to analyze cocaine samples. All tests you need to presumptively analyze cocaine.

The MMC Cocaine Analyzing Kit consists of:

- Cocaine / Crack test (3 tests)*
- Cocaine Purity Test (3 tests)*
- Cocaine Cutting Agents test (3 tests)*

PROFESSIONAL POCKET WEIGHING SCALE

We recommend this professional pocket weighing scale together with the Cocaine Analyzing Kit.



0.001 gr - 20 gr

95 x 33 x 133 mm



A closer look at the MMC Forensic Test Kits...



MMC has developed very specific narcotic tests in accordance with forensic standards.

The MMC Presumptive Forensic Narcotic Tests offer an opportunity to test for the presence of controlled substances in powders, tablets, liquids, resins and on surfaces.

These tests are very sensitive and require only Nano-grams of the substance to produce accurate test results.

Additionally, these tests can identify controlled substances dissolved in a liquid.

MMC has also developed a new buffer solution for the Cocaine Forensic Test.

At the request of a client in customs, we produced a cocaine forensic test that can detect cocaine which has been diluted in beverages such as Wine, Bitters, and Liquors.

The customer tested the buffer solution, which as they happily stated, worked perfectly.

Advantages of the MMC Presumptive Forensic Narcotic Tests are:

- Very sensitive and accurate
- Able to detect the presence of drugs in a sample at levels of <1 ppm (part per million)
- Specificity MMC FNT Tests Vs. GC-GC/MS: Negative: 99,38%, Positive: 97,45%
- No hazardous chemicals
- User friendly
- Highly cost effective in comparison to laboratory screening
- As reliable and accurate as laboratory screening
- No cross contamination as each test is supplied with a separate dropper of buffer solution (stick attached to cap)

A closer look at the MMC Forensic Test Kits...



Unscrew the white closure of the MMC special buffer solution container



Place a small part of the solid or tablet in the container of the MMC special buffer solution (*Cannabis (THC) / tablets, leave the bottle for 4-5 minutes before testing)



Dip the stick which is attached to the closure into the sample to be tested (liquid / powder)



Wet the cotton swab in the buffer solution and Rub the cotton swab a few times on the surface to be tested



Immerse the cotton swab in the container and mix the substance on the swab with the buffer solution by rotating the swab between the fingers



Close the MMC special buffer solution container tightly and thoroughly shake the MMC special buffer solution container for some seconds



Grasp the buffer solution container in one hand and snap off the tip so it becomes a dropper



Open the pouch of the single or multi 5 test cassette



Apply 4 drops* into the round opening of the cassette at 5 second intervals to achieve the best result

OTHER MMC PRODUCTS

COCAINE TRACE WIPES & SPRAY



The MMC Cocaine Trace Wipes presumptively detect the presence of Cocaine by simply wiping over the suspected surface. Test results are immediate. A blue result presumptively identifies the presence of Cocaine and may be sufficient cause for further investigation. These convenient Cocaine ID “Wipes” can presumptively identify cocaine from trace amounts of residue present on surfaces that have been in contact with cocaine. Each wipe is sensitive to trace amounts of residue; far less than could be collected for regular field testing. Each wipe is hermetically sealed in single-use foil envelopes. Unlimited shelf-life assures these wipes will provide dependable results when you need them. **Great for use by narcotics officers in the field and investigators at crime scenes.**



50 Wipes or in Bulk (800 wipes per carton) Size of wipe 70mm x 105mm



In close cooperation with Police and Customs & Excise authorities MMC International developed a handy & safe spray to identify cocaine impregnated in Textile, Cartons, Plastics and etc. but also traces of cocaine on all kinds of surfaces.

Advantages:

- Easy to carry with you (pocket size Height: 82mm - Diameter 18mm)
- Safe and easy to use
- No environmental risks (no aerosol gas!)
- Fast field test (result within seconds)
- Approx. 25 tests per ampoule (**250 tests per pack!**)
- Economic
- Applicator can be used multiple times

Packing: 10 x 2ml ampoules per box (including or excluding applicator) Applicator **separately available!**

OTHER MMC PRODUCTS

MMC STEROID CHECK



We developed Steroid Check especially to identify steroid substances. There are a lot of fake steroids available in the world market which could be very dangerous to use. Steroid Check is able to detect over 50 steroid substances.

Note that this test is ***not for testing urine samples.*** This test is ONLY to presumptively test the steroid substances.

Anavar, winstrol, boldenone, testosterone etc..



OTHER MMC PRODUCTS

ST-72 URINE STABILIZER TABLETS

MMC ST-72 Urine Stabilizer Tablets to avoid microbiological growth.

To secure forensic analysis for Alcohol, Drugs and Medical examinations.



General Information:

- ST-72 tablets for stabilizing urine for a period of 72 Hours.
- The tablets dissolve within approximately 60-90 minutes.
- Stabilize Urine Specimens For Delayed Laboratory Examination of Urine
- Maintains Bacterial Viability at Room Temperature Up to 72 Hrs.
- Eliminates refrigeration of urine culture specimens during storage or transit.
- Prevents over-growth of U.T.I. bacteria in specimen.
- Maintains Normal Growth and Chemical ID Characters of U.T.I. Bacteria
- Culture and Subculture growth characteristics are not altered.*
- Minimizes Growth of Patient Introduced Contaminants
- Accidentally introduced contaminants won't confuse results because levels are maintained for at least 24 hrs.
- False positives are virtually eliminated reducing the need for a second sample, and preventing needless patient anxiety.
- Convenient To Use
- No messy or strong smelling liquids to dispense into tubes.

Composition: Sodium Fluoride, Sodium Sulphate

Application:

ST-72 tablets are used by many Police Forces, prisons and other law enforcement agencies.

They are using urine-tainers to collect urine for analyzing for the presents of alcohol and drugs.

Each urine-tainer contains a MMC ST-72 tablet to stabilize the specimen as it will avoid microbiological growth for many weeks. The specimen can be send to the laboratory with no need for refrigerating the specimen. But for medical purposes the specimen should be refrigerated when send to the lab. The use of the ST-72 tablets will avoid inaccurate analysis lab results due to delay in storage or transport time of the urine specimen.

One (1) ST-72 tablet in a urine-tainer of approx. 27 - 30 ml. Storage: 5 - 35°C Stability 3 years after production date.

Available in collection tubes (48 per carton) or in bulk containers of approx. 2000 tablets.



Storage after usage: Place container in evidence bag. (Refrigerate the specimen for medical examination)

Warning: For in vitro use only. Do not ingest the tablets (MSDS available on request)

OTHER MMC PRODUCTS

MMC MOBILE NARCOTICS MINI LAB

Recently we developed a professional mobile narcotics mini lab to support the officer in their task of narcotics testing on location. The customer may compile the kind of MMC narcotics / precursor tests inside the mobile lab themselves. It is also possible to purchase the mobile mini lab without contents. And we as MMC could fill your mobile lab for other various applications such as: Forensic Evidence Collection (*Fingerprint, Post Mortem, Sex abuse*) or any other configuration.

Typical usage: Forensics, Military, Customs & Excise, Prisons etc...



Your own mini laboratory at any location.


*Dividers also
separately available*



OTHER MMC PRODUCTS MMC MOBILE NARCOTICS MINI LAB



BE PREPARED AT ANY CRIME SCENE!

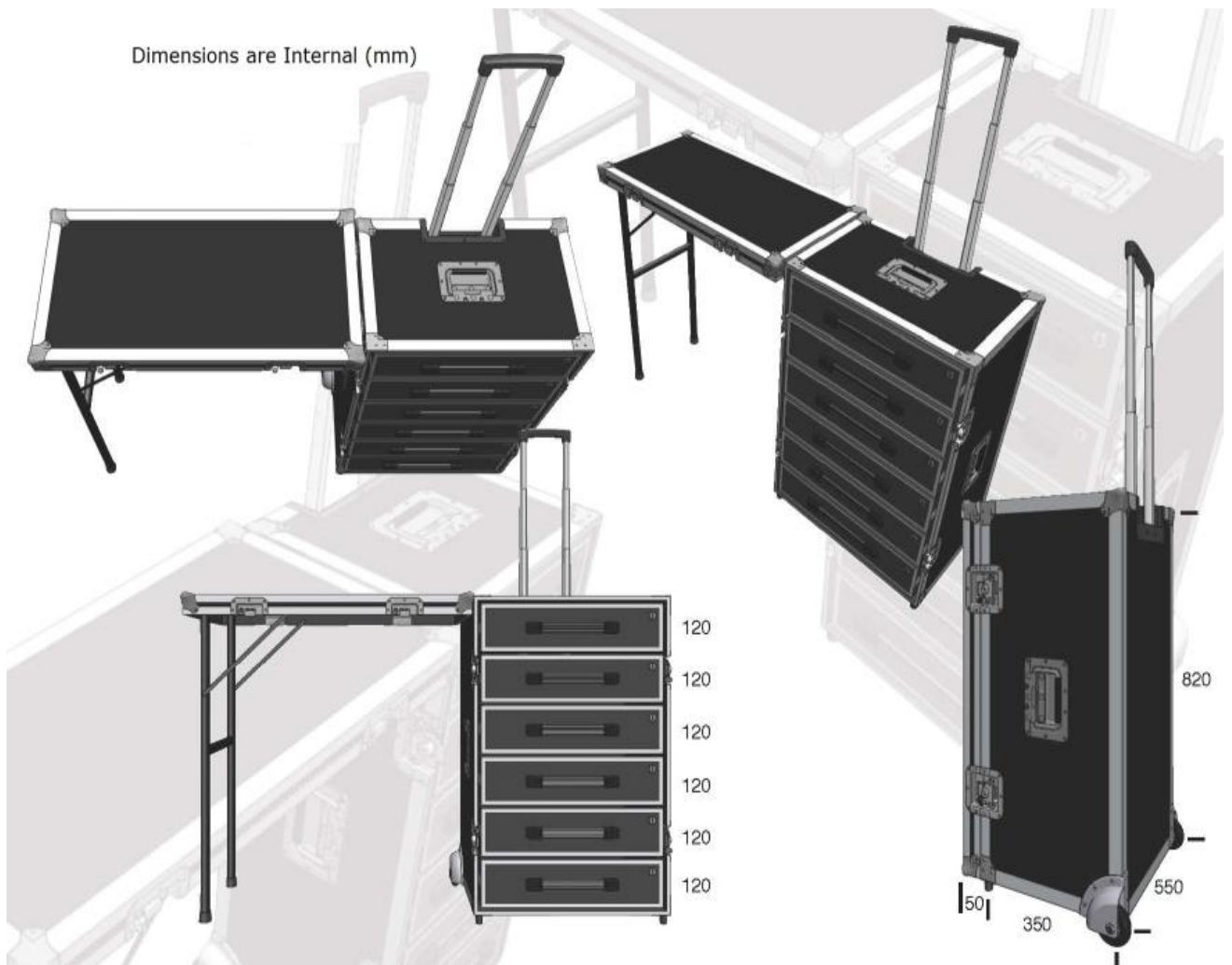

OTHER MMC PRODUCTS

MMC MOBILE NARCOTICS MINI LAB

Contents:

30 Packs MMC narcotic/precursor Field Tests (compiled by end user)
 1 Pack of MMC Cocaine cutting agents tests
 1 pack of 50 MMC Cocaine Trace Wipes
 50 Evidence bags small
 1 Large Leatherman Multi Tool
 1 Set of Marking Labels
 1 Stainless steel spatula
 2 Small hand drills 2mm + 4mm
 1 Desktop Magnifier with light
 50 Spatulas
 1 Notepad + pencil
 1 Laboratory wash bottle 250ml
 50 Alcohol cleaning wipes
 10 Pair of disposable shoe covers
 3 Multi urine Cup Test
 8 Liquid collection tubes
 20 GHB/GBL 1,4 Butanediol test strips
 1 Document holder

1 Pack of MMC Cocaine purity Field tests
 1 Pack of MMC Cocaine analyzing kit
 MMC Cocaine spray + applicator (+/- 25 tests)
 50 Evidence bags medium
 1 Mini Mag Lite
 50 Mini Pipettes
 1 Stainless steel scissor
 1 Pair of stainless steel tweezers
 2 Plastic Measuring Cylinder 25ml + 100ml
 1 Pocket Balance
 1 Five (5) mtr Measuring Tape
 1 MMC Narcotic Field Test ampoule rack
 1 Safety Goggles
 1 Pack of tissues
 5 Multi Saliva test
 100 Sealable bags 12x8 cm
 1 Five (5) meters of pH paper
 Digital Photo camera (with sd card and camera bag) Note: Camera may change without notice when it is not available.

1 Pack of MMC Heroin purity Field tests
 10 Sets of MMC Forensic multi Tests
 50 Pair of nitrile gloves (1 box size L)
 10 Evidence bags large
 5 Professional Dust masks
 10 Large Pipettes
 1 Small copper brush
 1 Permanent Marker
 50 Cotton Swabs
 1 Clip Board
 1 MMC ID Wall Chart
 1 Scalpel knife
 5 Disposable Overalls
 2 Sterile collection cups
 12 Collection tubes
 1 Scotch tape holder with 4 roles of tape
 10 ST-72 Tablets
 1 Flight case with 6 drawers



For other contents or colors of the case, please contact us...

OTHER MMC PRODUCTS MMC NARCOTICS EDUCATIONAL CASE

This **Narcotics Education Case** is a great help for narcotic officers, police academies and other Law Enforcement Agencies to provide a better awareness of illicit drugs.

The Narcotic Education Case holds four (4) trays each with eight (8) crystal cases. Each placebo drug comes in its own crystal case for extra protection. The trays and each crystal case are removable and can be passed around during your training session. In each case one (1) guide will be supplied.

In this guide you will find detailed information regarding the contents of each crystal case, such as street names, usage and effects, but also detailed testing procedures.

The Narcotics Education case also is equipped with wheels which you can easily click onto the case.



M.M.C. INTERNATIONAL

DRUG TRAINING COURSES

OTHER MMC PRODUCTS

TAMPER EVIDENCE BAGS

These bags are intended for use with confiscated drugs. Use the specialized plastic bags for handling narcotic evidence. These bags are serialized, have integrated tamper-proof seals, and detachable receipts. The serialized control number appears on both the bag and the detachable receipt. Each bag has a writable surface with blank fields to document all what is required.

- Serialized control number on all bags
- Removable receipt includes control number
- Integrated tamper-proof seal
- Writable surface for incident documentation (other remarks on the back)
- Ball point pen can be used to write on bag surface
- 1000 or 500 bags per carton (depending on size of bag)



6 x 9" / 15.2 x 22.8 cm, 9 x 12" / 22.8 x 30.5 cm, 12 x 16" / 30.5 x 40.6 cm (other sizes / artwork on request).

OTHER MMC PRODUCTS

VINYL GLOVES



MMC International also supplies Vinyl Gloves.

One (1) pair in a small bag
 10 pairs in outer bag
 50 outer bags per Carton
(Total of 500 pairs all separately packed)

OTHER MMC PRODUCTS

MMC TELESCOPIC INSPECTION CAMERAS

Telescoping inspection camera with strong LED Lighting and a 3.5" LCD Monitor allows to inspect hard to reach places such as containers, trucks, cars, tanks, ships, attics etc. for controlled substances / contraband. The unit comes in a strong case, including a rechargeable battery. The maximal inspection length is 315cm.



OTHER MMC PRODUCTS

MMC PROFESSIONAL DRUG SCALES



MMC INTERNATIONAL TRAINING COURSES

MMC International provides an online, no charge certification program for our customers.

Simply log onto www.narcoticstests.com and watch the instructional video. By following the step-by-step guidelines before taking the test, you can become certified in the use of the MMC Presumptive Testing Kits, Cocaine Trace Wipes, and Cocaine ID Spray.

After successful completion of the online exam, a Certificate of Completion for temporary use will be e-mailed to you and a hard copy suitable for framing or for use in court will be mailed separately.

MMC also organizes very unique courses in the field of controlled substances.

The courses are especially meant for:

- * Drug Recognition Experts (DRE)
- * Customs Officers
- * Airport Security Officers
- * Border Police
- * Prison Officers
- * Traffic Police
- * Railroad Police
- * Port/maritime customs



We can provide you a very good quotation, please send us the modules you are interested in, the location and how many participants.

[Module 1: Narcotics Recognition, Side Effects and Testing procedures](#)



This **(1) one-day** course is designed to provide detailed information related to controlled substance recognition, side effects of using drugs and field testing procedures.

Topics include:

- Recognition of controlled substances
- What are the side effects of using controlled substances
- Field testing procedures

[Module 2 Part I: Growing treatment and safe dismantling of cannabis \(Marijuana\) plantations.](#)



This **(1) one-day** course is designed to provide detailed information related to Growing treatment and safe dismantling of cannabis (Marijuana) Plantations. Students will learn ways to identify the most important narcotics on the market today.

Topics include:

- Growing treatment and safe dismantling of cannabis (Marijuana) Plantations
- Recognition of cannabis plantation from the outside
- Precautions to take before entering the plantation / location
- Recognition of Equipment used to grow cannabis
- Learn what you need to report
- Identify what kind of cannabis plantation it is (Mother plants, cuttings (M/F plants)
- Advise: Safe dismantle procedures
- Legal aspects



MMC INTERNATIONAL TRAINING COURSES

Module 2 Part II: Take away illegally acquired profits



This course is designed to learn how to deal with illegally acquired profits with help of case simulations.

Case A:

Pre-operation planning on how to start an investigation into a drug dealer.
How to calculate illegally acquired profits of a drug dealer.

Case B:

How to start an investigation on a cannabis plantation
How to calculate illegally acquired profits of a drug dealer.
Legal aspects

Module 3: Recognition of production and the method of synthetic drugs



This **(1) one-day** course is designed to learn the precautions of dangerous issues related to production facilities and methods of synthetic drugs

Topics include:

- Precautions to take before entering the facility
- Recognition of Equipment used for the production of synthetic drugs
- Production method information
- Advise: Dismantle procedures
- How to make Cocaine
- Cutting Drugs
- Legal aspects

Module 4: Smuggling techniques.



This **(1) one-day** course is designed to learn various smuggling techniques.

Topics include:

- Hiding methods of controlled substances
- Sampling & Testing
- Legal aspects



Officers in training

One of our professional instructors in action...



Airport Ground Support Equipment



Mobile Laboratory



Mobile X-Ray baggage scanning truck



Mobile Drug Incinerator



Contraband Detector



Safety Cabinets



Security Lockers



Alcohol Breath Analyzers



Many other Law Enforcement Products



DOA Saliva Tests



DOA Urine Tests



WWW.NARCOTICTESTS.COM



**MMC International BV Frankenthalerstraat 16/18
4816 KA Breda The Netherlands**
Phone: +31-76-5711140 Fax: +31-76-5719300
info@mmcinter.com www.narcotictests.com